


Rectal Swab Instructions

1

Label container

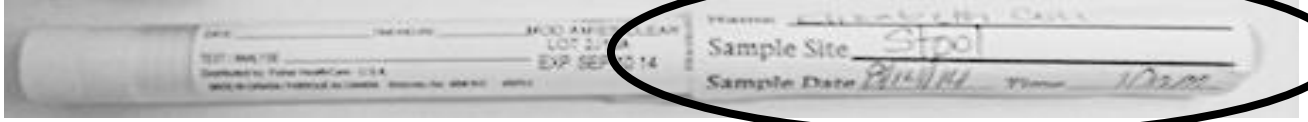

(your name, date of birth, date & time of sample)

2 Take white inside cap off and throw away in trash can



3 Take blue swab and insert INSIDE rectum 1-1.5 inches (gently rotate)



4

MAKE SURE THE TIP IS BROWN!



(If it is not get another swab and start from step 1)

5

Insert Swab in tube




(make sure it touches gel in bottom)

Close blue lid attached to swab tightly by pushing down firmly

6

Wash hands

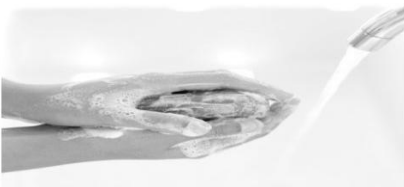

7

Return closed tube to designated person



shutterstock - 191557223